



Ultima Dental Systems

3 Way Syringe

Button Kit Install Instructions




This manual applies to all Ultima Dental units:


Ultima 250, 500, 750, and 2000 Series.

3 Way Button Repair Install Instructions

1. Remove all pressure from the dental unit
2. You will need to remove the pin on the side of the 3 way syringe in order to release the buttons. Use a small hex key to push and slide out the pin. *(Tip) *If you do not have access to a set of hex keys a sturdy paper clip should do the trick**

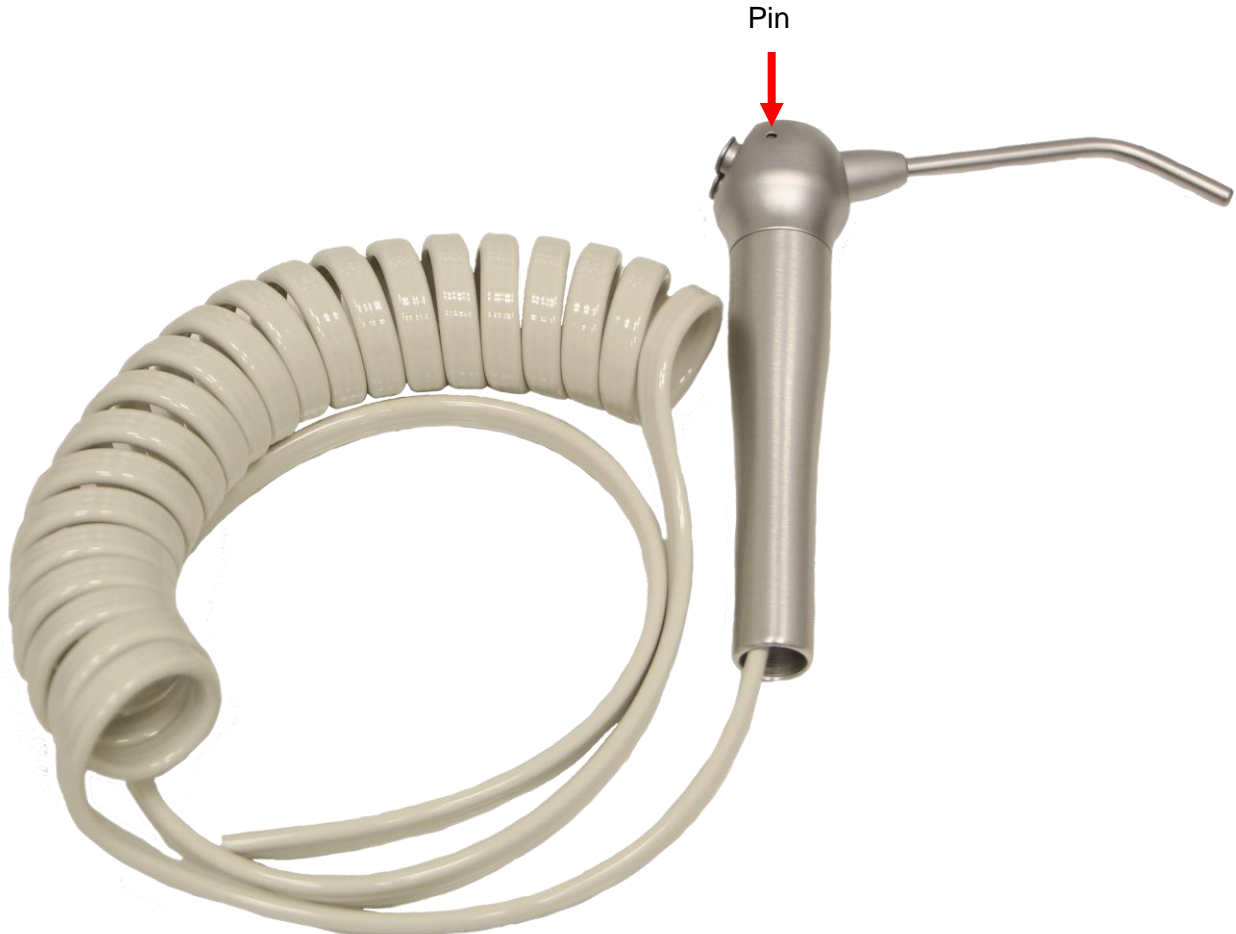

3. To remove the pin you must press and hold down both buttons in order to push/slide the pin through.
4. Once the pin is removed, pull both buttons out and thoroughly clean inside the 3 way syringe head where the buttons are housed. You may use Q-tips and rubbing alcohol.

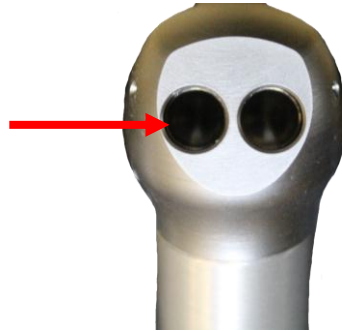

5. After cleaning the inside of the 3 way syringe install the new buttons. It does not matter which button goes where, they are both the same.



6. Press and hold down both buttons to slide the pin back inside the 3 way syringe.
7. Connect the dental unit and allow the compressor to build pressure. Once the unit is ready for use test the 3 way syringe.

For further assistance contact the UDS Technical Support Team:

Call (954) 772-9779 or Email us at UDS.FLA@Gmail.com