# Ketosis Management, in the palm of your hand.



#### The gold standard for ketone testing...

The *CentriVet*<sup>™</sup>GK Blood Glucose & Ketone Monitoring System is a *dual meter* that measures both blood glucose and blood ßHBA, giving you accurate results in seconds.

## Early Detection of Subclinical Ketosis means economic loss protection...

The *CentriVet*<sup>™</sup>GK Blood Glucose & Ketone Monitoring System is a diagnostic tool that can be used for upstream assessment of subclinical ketosis in dairy cows. Detecting dangerously high blood BHBA concentrations can prevent economic losses from decreased milk yield, decrease chances of removal from herd, and allow for quicker adjustment of nutrition plans.

- Calibrated for Bovine Blood
- Accurate Results in Seconds
- Dual Meter for Glucose & Ketone Testing
- Priced for Profit

Glucose and Ketone Testing Now Available for:










www.centrivet.com



# Technology Used By Vets, Now Available for Farm Use.

- Accurate, quantitative results comparable to reference method\*
- Strip Port Ejector for convenience and safety
- Big, bold screen for easy viewing

<i>CentriVet</i> ™ GK Blood Glucose & Ketone Meter, Bovine Blood Sample			
System Accuracy Results for D-3-Hydroxybutyrate Concentration ≥ 1.5 mmol/L and < 1.5 mmol/L			
Within ±15% or ± 0.3 mmol/L	Within $\pm 20\%$ or $\pm$ 0.3 mmol/L		
321/336 (95.5%)	331/336 (98.5%)		

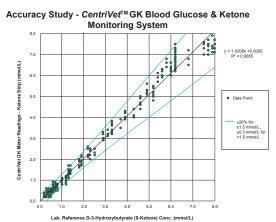

## **Full Specifications**

Feature	Specification	
Sample Size	Glucose: 0.8 µL	
	ß-Ketone: 1.2 µL	
Test Time	Glucose: 5 seconds	
Test Time	ß-Ketone: 10 seconds	
Test Pango	Glucose: 0.6 – 33.3 mmol/L (10 – 600 mg/dL)	
Test Range	ß-Ketone: 0.0 – 8.0 mmol/L	
HCT Range	20-70% for both Glucose and ß-Ketone	
Memory Storage	450 blood glucose and ketone records with result type, date, and time	
Test Averaging	7, 14, 30, 60, and 90- day averaging for glucose, before and after meal markers for glucose and ketone testing	
Battery Life	3,000 tests	
Operating Temperature Range	5 - 45°C (41-113°F)	
Operating Humidity Range	10 – 90%	
Automatic Shutoff	2 minutes of inactivity	
Power Source	2 CR 2032 3.0V coin cell batteries	

## **Ordering Information**

Product	Contents	ltem Code US	Reference No. International
<i>CentriVet</i> <sup>™</sup> GK Blood Glucose & Ketone Meter Kit	Meter, Batteries, Carrying Case, User's Manual, Reference Materials, Warranty Cards	781211	GK114-13C
<i>CentriVet</i> <sup>™</sup> Blood Glucose Test Strips	50 Strips/vial	781222	GK134-12C
<i>CentriVet</i> <sup>™</sup> Glucose Control Solution	2 x 2 mL bottles	781232	GK124-13C
CentriVet <sup>™</sup> Blood Ketone Test Strips	25 Strips/vial	781221	GK134-10C
CentriVet <sup>™</sup> Ketone Control Solution	2 x 2 mL bottles	781231	GK124-12C

\*Additional data and instructions can be found in the package insert

ACON Laboratories Inc., San Diego, CA, 92121 PN 2160095002

#### www.centrivet.com

MAI Animal Health - UNITED STATES 1.800.447.0687 sales@maianimalhealth.com MAI Animal Health - EUROPE +44 (0) 1225 791 184 sales.uk@maianimalhealth.com MAI Animal Health - AUSTRALIA +61 (0) 2 6372 6622 sales.au@maianimalhealth.com

MAI Animal Health - NEW ZEALAND +64 (0) 9 444 2115 sales.nz@maianimalhealth.com

